

THE **Stealth S.S.A.**[®]

SECRET SOUND AMPLIFIER

Instruction Booklet

PLEASE NOTE: The Steath S.S.A.® personal sound amplification product is intended to amplify environmental sound for non-hearing impaired users, and is not intended to compensate for hearing impairment. If you believe that your hearing is impaired, you should consult with a licensed physician.

IMPORTANT:

Before using the unit for the first time, you must charge the internal rechargeable battery for four hours. Please refer to **Page 6.** of this guide, the section titled ***Recharging the Battery***, for assistance.



PRODUCT SPECIFICATIONS:

- Power Supply: Rechargeable Battery
- Voltage: 3.7VDC,
- Operating Current: $\leq 16\text{mA} \pm 20\%$
- Speaker Impedance: 200 Ohm
- Output Power: 3mW
- Unit Weight: 20g (including built-in rechargeable battery)

Table of Contents

- 2. Product Specifications
- 3. Contents of Package
- 4. Welcome to the Stealth S.S.A.[®]
- 5. On/Off
- 6. Rechargeable Battery
- 6. Recharging the Battery
- 7. Volume Control
- 7. How to Wear and Adjust Ear Amplifier
- 8 - 10. Frequent Questions
- 11. Ordering Replacement Parts
- 12. Refund and Return Policy

Welcome to the Stealth Revolution!

You are now the owner of the Stealth S.S.A.[®], the revolutionary sound amplification system that is lightweight, compact, and discreet. When used correctly, the Stealth S.S.A.[®] promotes a happier, healthier lifestyle by allowing you to hear while maintaining your normal routine and activities. Please read the following instructions through carefully before using.

Thank you for purchasing the Stealth S.S.A.[®]

© 2015 KAGAN Unlimited

Stealth S.S.A.[®] instructions and its related logo are
registered trademarks of KAGAN Unlimited
Chicago, Illinois, 60625

CONTENTS OF PACKAGE

- (1) Stealth S.S.A.[®]
- (3) Soft Ear Piece Covers
- (1) AC Charger, Rechargeable Lithium Batteries

Directions for Use:

*Remember to charge the Stealth S.S.A.[®] for a minimum of 4 hours prior to first use!
See section titled "Recharging the Battery" on the following page for help.*

SWITCH "ON" THE UNIT

The ON-OFF Power switch is located on one side of the unit. When you slide the switch to ON, the green light comes on. Slide it to the OFF position and the green light turns off. Be sure to turn the unit OFF when you recharge. The battery cannot be recharged when the unit is ON.

RECHARGEABLE BATTERY

A rechargeable lithium battery operates this unit. *Before* using the unit for the first time, you must charge the internal lithium rechargeable battery for *four hours*. If the sound is weak while the volume control is at the maximum position, the battery should be recharged immediately.

RECHARGING THE BATTERY

The AC adapter included with this unit is for recharging the built-in battery. Locate the DC jack at the larger end of the unit. To recharge the battery, insert the DC plug into the DC jack and plug the AC adapter into a wall socket. A red light will come on to indicate that the battery is being charged. The red light will turn off when the battery is fully recharged. *Only* use the AC adapter that has been supplied by the manufacturer. Damage by unauthorized adapters will not be serviced or refunded.

VOLUME CONTROL

Volume control is located on the side of the unit. Turn the knob towards the plus (+) symbol to increase the volume. Turn the knob towards the minus (-) symbol to decrease the volume. Once the volume control is set at the desired level for your personal listening, you need only slide the power switch to the ON position and you can enjoy the same sound level each time.

HOW TO WEAR THE EAR AMPLIFIER

Wrap the rubber wire over your ear until the unit is secure and comfortable. This rubber wire is made to easily adjust to all ear shapes and sizes. If desired, the soft ear tip cover can be taken off and cleaned. Use lukewarm water only on the soft ear tips. Do not submerge the Stealth S.S.A.[®] in water.

CAUTION: Only use the AC adapter that has been supplied by the manufacturer. The manufacturer does not refund or service items damaged by unauthorized adapters.

FREQUENTLY ASKED QUESTIONS:

1) Why do I need to charge the battery before the first use?

The Stealth S.S.A.[®] uses a lithium battery. These batteries require a full charge before the first use of the product for the batteries to function at their optimal use. The Stealth S.S.A.[®] can be charged for 4 hours or overnight. The red light (seen while charging) will turn off when the unit is completely charged.

2) How do I know if the battery is fully charged?

While the Stealth S.S.A.[®] is being charged, there is a red light on the unit. When the red light goes off, the battery is fully charged. (See page 6 for details on charging your unit.)

3) Will the light turn green when battery is fully charged?

No, the light will turn off when fully charged and the unit is in the off position. When the unit is turned on again, the green light will re-appear on the unit.

(F.A.Q'S CONTD.)

4) How can I get it to fit in my ear properly?

The Stealth S.S.A.[®] comes with three different style ear tips. Start with the smallest tip and see how the unit fits. If the unit is loose, try the next larger size tip until you find a tip that fits comfortably in your ear. Use the flexible foam earpiece to wrap around the outer ear to hold the unit at the desired position.

*5) How do I clean the **Stealth S.S.A.[®]** ?*

You can wipe the unit with a clean wet cloth. Do not immerse the unit in water or it will destroy the battery and possibly cause electric shock.

*6) How do I clean the **Stealth S.S.A.[®]** soft ear tips?*

You can wash the soft ear tips in a bowl of warm water and wipe clean with a cloth.

7) *Why is there a high pitched noise?*

Audio feedback may be caused when the Stealth S.S.A.[®] earpiece is exposed to its own super-sensitive microphone. Feedback is normal, however, repeated and/or prolonged occurrences should be avoided. To prevent audio feedback, remember to switch OFF the Stealth S.S.A.[®] before placing it into, or removing it from, your ear.

8) *Why is the unit not holding a charge?*

The Stealth S.S.A.[®] will hold a charge up to 10 hours. If your unit is not holding the charge, it could be because it was not properly charged the initial four hours prior to the first use.

9) *Will the **Stealth S.S.A.[®]** work with my cell phone?*

The Stealth S.S.A.[®] is an audio amplification device. While the unit is cleverly disguised as a cell phone earpiece, it is not intended to be compatible with any phones on the market and does not work in conjunction with cell phones.

REPLACEMENT PARTS

Additional Ear Tips: \$2.00/pack of three (includes postage)

Battery Charger: \$19.95/charger plus \$6.95 UPS, total: \$26.90

Stealth S.S.A.[®]: \$39.95/unit plus \$6.95 UPS, total: \$46.90

To request replacement parts, include a check or money order for the appropriate amount payable to KAGAN Unlimited with a letter specifying the item you require to:

Stealth S.S.A.[®] *Replacement Parts*
c/o KAGAN Unlimited
3336 West Lawrence Avenue, Suite 301
Chicago, IL 60625

Please Note: Requests submitted without proper payment will not be fulfilled. Please allow 4 weeks for processing all requests for replacements and replacement parts.

REFUNDS & RETURNS

To receive a credit or a refund for faulty Stealth S.S.A.[®] unit, you must return that unit, with proof of purchase, *where the product was purchased*. Only the company that sold you the product is able to refund or credit your account. The authorized sellers of the Stealth S.S.A.[®] have the right to refuse a refund at their sole discretion. Please refer to your proof of purchase, packing slip, or receipt to determine where you should return your product and the specific authorized dealers statements and policies.

To request a replacement for a Stealth S.S.A.[®] unit that has been purchased by you from the manufacturer, please return the unit with proof of purchase and a letter stating the reason for your return to:

Stealth S.S.A.[®] Replacement Parts
c/o KAGAN Unlimited, 3336 West Lawrence Avenue
Chicago, IL 60625



Made in China.

© 2015 KAGAN Unlimited

Chicago, IL 60625

All Rights Reserved.